DISCO / BLOCCHETTI IN ZIRCONIA

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ZIRKONIUM DISCS / BLOCKS

- Isostatically pressed, pre-sintered
- INSTRUCTIONS FOR USE**

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
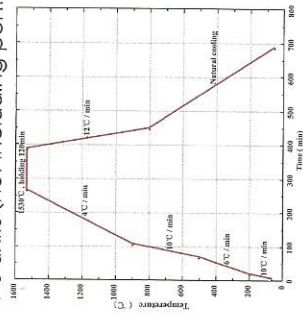
DE Healthcare Ltd
Unit 9, Kingshorpe Business Centre,
Studdard Road, Kingshorpe,
Northampton NN2 6NE
Denver, PA 17517 USA

Sintering temperature:


1-5 units (not including pontics)

2-5 units (including pontics)

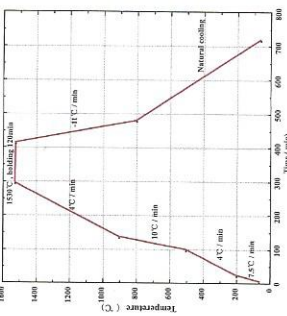6-9 units:

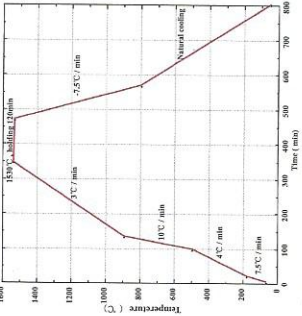Min °C: 1.450

INSTRUCTIONS FOR USE

1. Fix the blank / block according to the CAM-System instructions. Set the engagement-factor into the unit.
2. Start the milling unit.
3. After milling, extract the blank / block with milled restoration.
4. Remove the restoration from the attached blank / block with a fissure bur.
5. Remove any dust from the restoration with pressured air before sintering.
6. Position the restoration on the sinter based on the furnace tray.
7. Program the Zirlux STI according to the sintering instruction
8. Run the sintering cycle.
9. Wait until cool down is completed before removing the sintered restoration.
10. Inspect the restoration in respect of flaws, wall thickness, seat and marginal fit. If necessary, make small adjustments with a water-cooled diamond bur.
11. If it is now ready for veneering, the framework is now ready for veneering / anatomical crown is ready for glazing!

- Safety precautions**
- Design the restoration to maximize cross-sections.
 - Carefully read the MSDS.
- Sinter instructions**
- | Ramp Temp. °C/h | T. max °C | Dwell Time h | Cooling |
|-----------------|-----------|--------------|-------------------------|
| 600 | 1450-1550 | 2 | Natural furnace cooling |
| 300 | 1450-1550 | 2 | cooling |

Indications for use

Zirconium discs / blocks are indicated for use as anatomy crown and a substitute for porcelain fused ceramic fixed dental restorations. Limitations are listed in table 1.

Table 1: Indications of use and maximum number of pontics.

Process chain	Single unit crowns		Bridges, number of pontics	
	Anterior	Posterior	Anterior	Posterior
Press blank/ machined blocks to enlarged automatic crown, milled and veneered	X	X	3	2

*X in brackets, digits show maximum number of pontics.

Limitation of Bridge, number of Pontics
All Zirlux-blank / blocks are solely for use by or on the order of a dental professional. They are not for use by the general public or over-the-counter.

Contraindications

Contraindications are all applications not indicated in table 1 above.

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